



Model: 9504 Probe Extension 12 cm

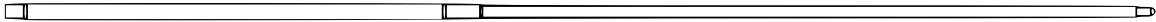

Model: 9505 Probe Extension Ball Tip, 12 cm

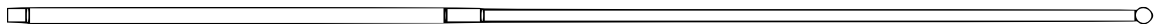

Model: 9506 Probe Extension 4 cm



Probe Extension for Checkpoint Stimulators

INSTRUCTIONS FOR USE

Model: 9504 / 9505 / 9506

Probe Extension for Checkpoint Stimulators

INSTRUCTIONS FOR USE

INDICATIONS FOR USE:

The Checkpoint Guardian Probe Extension Family is a single-use, sterile medical device accessory family intended for use in conjunction with a Checkpoint Stimulator to provide electrical stimulation of exposed motor nerves or muscle tissue to locate and identify nerves and to test nerve and muscle excitability.

PRECAUTIONS:

- Do NOT use any Probe Extension that does not press firmly onto the Checkpoint Stimulator probe or fails to connect electrically to the Stimulator.
- Do NOT use the Probe Extension if any metal surfaces other than the tip are exposed.
- This device is an accessory to a Checkpoint Stimulator Locator (P/N 9094, 9095, 9394), and all contraindications, warnings, and precautions associated with the Stimulator described in its manual are applicable.
- Single use only.

INSTRUCTIONS FOR USE:

1. Prepare the Checkpoint Stimulator

- Refer to your Stimulator's manual for preparation and use instructions.

2. Prepare the Probe Extension

- Remove the sterile pouch from the box.
- Inspect sterile pouch for any damage.
- Do not use if sterile pouch or product appears damaged.
- Remove the Probe Extension on its backer card from the sterile pouch.
- Remove the Probe Extension from the backer card.



3. Connect the Probe Extension to the Checkpoint Stimulator

- Connect the connector end of the Probe Extension to the stimulating end of the Stimulator.



4. Stimulate

- The same stimulation technique used with a Checkpoint Stimulator on its own should be used with the Stimulator/Probe Extension combination. The Probe Extension wire may be bent, removed, and reattached as needed.

Additional notes:

- Refer to your Checkpoint Stimulator's manual for Stimulator setup, use, and indicator light meanings.

End of use / disposal:

- Disconnect the Probe Extension from the Stimulator probe and dispose of it in an appropriate biohazard container.








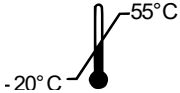

WARRANTY and LIMITATIONS:







Checkpoint Surgical, INC. (CPS) warrants that this product has been manufactured, packaged, and tested with reasonable care and will be free from defects in workmanship and materials. CPS further warrants that the product will remain sterile for a period described on the product's label, provided the original packaging remains intact. This product is for single-use only and is not intended or designed for reuse. This warranty shall not apply to product that has been re-sterilized, repaired, altered, or modified in any way, or to products that have been improperly stored or operated. CPS will not be liable for any incidental, special or consequential loss, damage, or expense resulting, directly or indirectly, from the use of the product. The sole obligation of CPS shall be to refund or replace, at its option, any device that CPS determines was defective at time of shipment if notice thereof is received before expiration date described on such product label. Buyer assumes all liability, whether based upon warranty, contract, negligence, or otherwise, for damage resulting from the handling, possession, use or misuse (including reuse) of this product. Because CPS has no control over the operation, inspection, maintenance, or use of its products after sale and has no control over selection of patients, THIS WARRANTY IS EXPRESSLY IN LIEU OF ANY OTHER EXPRESS OR IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR ANY PARTICULAR PURPOSE, AND OF ANY OTHER OBLIGATION ON THE PART OF THE SELLER. The remedies set forth in the Warranty and Limitations shall be the exclusive remedy available to any person. No agent, employee, or representative of CPS has any authority to change any of the foregoing or assume or bind CPS to any additional liability or responsibility in connection with this warranty. Buyer's use of this product shall be deemed acceptance of the terms and conditions of this Warranty and Limitations.

CHECKPOINT (P/N 950X) SPECIFICATIONS:

	9504	9505	9506
Overall Length	157.89 mm	157.65 mm	76.58 mm
Contact Surface Area	7.9mm ²	14.96 mm ²	7.9mm ²
Contact Material	304 Stainless Steel	304 Stainless Steel	304 Stainless Steel
Connection Type	Compatible with Checkpoint Stimulator/Locator probe	Compatible with Checkpoint Stimulator/Locator probe	Compatible with Checkpoint Stimulator/Locator probe
Operating Temperature	+16°C to +26°C (61°F to 79°F)	+16°C to +26°C (61°F to 79°F)	+16°C to +26°C (61°F to 79°F)
Relative Humidity	30% to 75%RH	30% to 75%RH	30% to 75%RH
Atmospheric Pressure	70kPA to 106kPa	70kPA to 106kPa	70kPA to 106kPa
Storage Temperature	-20°C to +55°C	-20°C to +55°C	-20°C to +55°C

- Product has no user serviceable or repairable parts.
- Sterilized with Ethylene Oxide.
- Do NOT re-sterilize.

	<p>EN Model Number</p>
	<p>EN Caution, consult accompanying documents</p>
	<p>EN Do not reuse</p>
	<p>EN Sterilized using ethylene oxide</p>
	<p>EN Lot number</p>
	<p>EN Use by day DD of month MM of year YYYY</p>
	<p>EN Keep dry</p>
	<p>EN Storage Temperature LOW-HIGH -20°C to +55 °C</p>
	<p>EN Manufacturer</p>

	<p>EN Do Not Use If Package Is Damaged</p>
	<p>EN Refer to instruction manual/booklet.</p>
	<p>EN Non-pyrogenic</p>
	<p>EN This system does not contain natural rubber (including natural rubber latex, dry natural rubber, synthetic latex and synthetic rubber that contains natural rubber in its formulation)</p>
<p>Rx only</p>	<p>EN Caution: U.S. Federal Law restricts this device to sale by or on the order of a physician or properly licensed practitioner.</p>
	<p>EN Probe Extension requires a Checkpoint Surgical stimulator.</p>
 <p>*+B13095052/ \$\$3yyMMddY3XX1*</p>	<p>EN UDI barcode with Checkpoint LIC, part number, unit level (1: sterilized in pouch, 2: single unit sales carton), expiration date and lot number.</p>



Checkpoint Surgical, Inc.
6050 Oak Tree Blvd.
Independence, Ohio 44131
Tel: 216-378-9107
www.checkpointsurgical.com

© Checkpoint Surgical, Inc.
August 2021